

INSTALLATION INSTRUCTIONS

ASSIST SPECIAL NEEDS

TOILET SEAT

PRODUCT CODE Assist Special Needs Toilet Seat - Blue	RB6015B
Assist Special Needs Toilet Seat - Grey	RB6015G
Assist Special Needs Toilet Seat - White	RB6015W



The Assist Toilet Seat is supplied in Blue or Grey for AS1428.1 installation compliance.


The seat is also available in White for assisted living/geriatric applications.

Lateral stabilisers are incorporated in the design of the seat

Seat mounting is vandal resistant.

INSTALLATION STEPS

1. Fit hinge mount assembly as shown into the seat fittings (Note: make sure to fit the hinges the right way round - the elbow posts should be to the back).
2. Fit the dress and centre washers and assemble through the seat bolt holes in the WC.
3. Adjust seat so that it fits into position on the bowl. Fit the lateral stabilisers so that they are located correctly just on the inside of the bowl rim. Fit the stabiliser cover pads.
4. Tighten seat bolts from underneath using the spanner tool provided

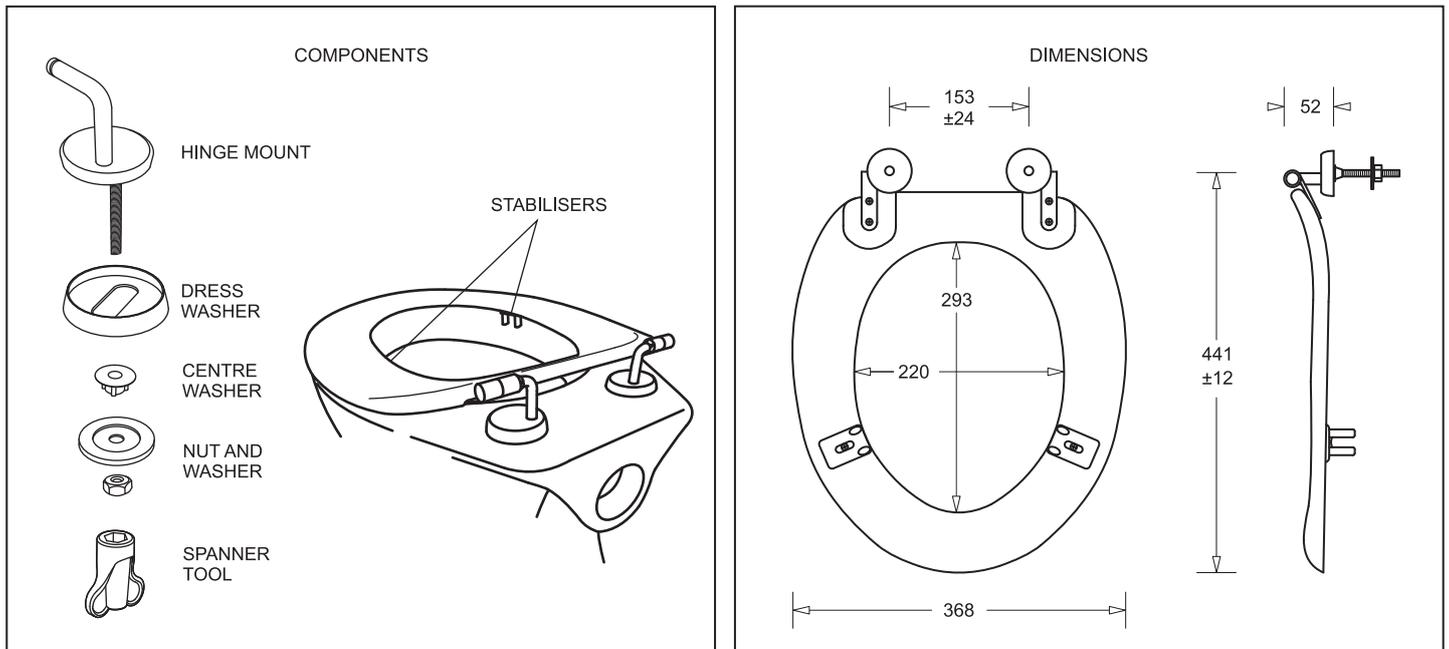

CARE

1. Clean only with a soft cloth using mild soapy warm water
2. Do not use solvent based chemicals
3. Do not use harsh or abrasive cleaners

All installations should be carried out by a licensed plumber in compliance with the Plumbing Code of Australia and any State or Local Authority Regulations